

Shutdown System

### Tablet buttons

Start Tablet



Restart VSOL



Use only when requested

VSOL Toolkit





Control Centre



Resolution for most common software issues (+automatic support notification)

### Updates / service / internet


- The system is equipped with a working **4G** LTE connection, which provides an uninterrupted internet connection for syncing, backup, email and remote support
- Next to 4G, both a LAN and Wi-Fi connection are available
- Software updates are applied **automatically**
- Your images are **safely** stored on connect.vet
- To connect to your wifi, **NEVER** use the wireless adapter named XDETECTOR, always use the adapter named FREE

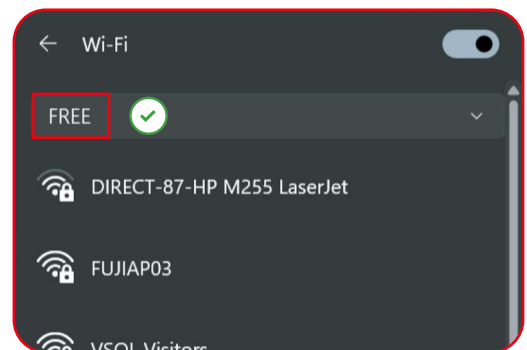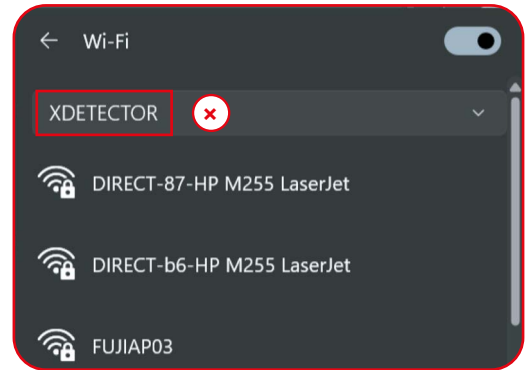

### USB-C detector charger

- The USB-C charger is **not** interchangeable between the the detector and the Windows tablet: **use the USB-C charger ONLY for the detector, use the dedicated charger for the Windows tablet.**
- The detector has a 6 hour autonomy
- The detector will go into **sleep** mode after 10 minutes and will auto **shut down** if not used for 30 minutes
- Charging the detector for 10 minutes results in 20 minutes work

More  
on our  
software

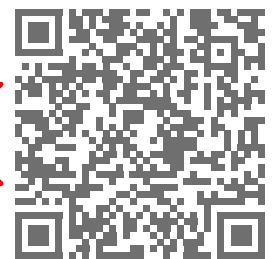

### Contact us now

UK +44 (0) 1506 460 023 IE +353 (0) 42 932 0070

support@imv-imaging.com

Follow us 

 [veterinarysolutions.be](http://veterinarysolutions.be)